

Test Details

Sample Type: Dry Blood Spot
Collection Location: At Home



The IgG Food MAP with Candida and Yeast analyzes a wide variety of foods, herbs, and spices common in Western, Asian, and Mediterranean diets, as well as Candida and yeast. It assists in the diagnosis of food sensitivities.

Key Details

- Talk to your healthcare provider if you are taking blood-thinning medications or have a blood clotting disorder.
- **Drink at least 16 oz of water** a few hours prior to collection.
- View collection instructions for successful sample examples
- Keep your collection card cool and dry until shipping.
- Ship your sample Monday through Thursday.

Before you Start

Scan QR code located inside your test kit or visit MosaicDX.com/register to register your test, so we know it belongs to you.


Collection Preparation

- A few hours before collection, drink at least 16 oz of water.
- Set in a quiet place away from pets, dust, cooking, make-up, or other potential contaminants.

Once Collected

- Avoid touching or smearing blood spots while handling.
- Do not stack the collection card or allow it to touch other surfaces while drying.
- Allow 3 hours for the sample to dry before shipping.
- Keep away from sunlight or heat sources which may affect the quality of the sample.

Scan QR code for step-by-step instructions

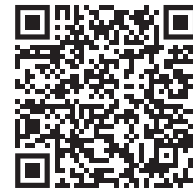

Any Questions?

If you have questions about any aspect of the specimen collection or shipping process, please feel free to contact us:



Phone | our friendly customer service team is available Monday through Friday 8am-5pm CST at 800-288-0383



Email | CustomerService@MosaicDX.com





MOSAIC[™]
DIAGNOSTICS
Formerly Great Plains Laboratory

All trademarks are owned by the company and its affiliates.
©2023 Mosaic Diagnostics. All Rights Reserved. MDX-Inst-Sheet-IgGDBS v.1 10-1-23

Visit [MosaicDX.com](https://www.mosaicdx.com) for more resources

(800) 288-0383 customerservice@mosaicdx.com
8400 W 110th Street, Suite 500, Overland Park, KS 66210

