

## Test Details

**Sample Type:** Urine

**Collection Location:** At Home



The TOXDetect Profile provides a comprehensive assessment of exposure to environmental toxicants.

## Key Details

- Do not collect your sample while menstruating.
- You will collect your sample **upon waking, before eating or drinking.**
- **Freeze your sample** before shipping.
- Ship your sample Monday-Thursday.  
For **international shipments**, ship Monday-Wednesday.

Scan QR code for  
step-by-step  
instructions



## Collection Preparation

- No provocation or discontinuation of items is required before collection. Please consult your practitioner for your personalized care plan.
- Place your ice pack flat in your freezer, so it will be ready to ship with your specimen.
- Review included shipping instructions.
- **Complete the barcode sheet** from your test kit, affixing a barcode sticker to each sample
  - Verify that the information is legible and includes the collection date and a matching name for the test order.
  - **Samples lacking a complete name and labels will be rejected.**

## Any Questions?

If you have questions about any aspect of the specimen collection or shipping process, please feel free to contact us:



**Phone** | our friendly customer service team is available Monday through Friday 8am-5pm CST at 800-288-0383



**Email** | [CustomerService@MosaicDX.com](mailto:CustomerService@MosaicDX.com)



**MOSAIC<sup>™</sup>**  
**DIAGNOSTICS**  
*Formerly Great Plains Laboratory*

All trademarks are owned by the company and its affiliates.  
©2023 Mosaic Diagnostics. All Rights Reserved. MDX-INST-SHEET-TOX 4-2024 v.1

**Visit [MosaicDX.com](https://www.mosaicdx.com) for more resources**

(800) 288-0383 [customerservice@mosaicdx.com](mailto:customerservice@mosaicdx.com)  
8400 W 110th Street, Suite 500, Overland Park, KS 66210

